



Positioning & Product Guide

Contents

Cushion Range

Microbead Cushions (PosiPerle/PosiPillow/Carewave)	Page 1-7
Fluidised Positioners (Flo-Form)	Page 8-9
Bamboo and Cotton Covers	Page 10

Positioning Guide

Head Neck & Ear Support	Page 11-14
Chest & Side Lying Support	Page 15-17
Arm & Hand Support	Page 18-19
Leg Support	Page 20-21
Leg & Knee	Page 22-24
Knee Support	Page 25-27
Knees & Abduction	Page 28-31
Foot & Heel	Page 32-36
Stabilegs & Foam Through Leg Cushion	Page 37-39
Kit Bags	Page 40-41
Positioning Examples	Page 42-45
PneumaPure Filter Technology	Page 46-47
Flo-Form Fluidised Gel Positioners	Page 48-49
Neonatal Fluidised Positioners	Page 50
Weighted Tail Cushion Covers	Page 51-52
KomfiTilt	Page 53

T: 01264 366330

E: info@pressurecaremanagement.com

W: pressurecaremanagement.com

PosiPerle & PosiPillow

Microbeaded Positioning Cushions

PneumaPure Filtration Technology

Proven Infection Control

Multi Patient Use

HOW IT WORKS?




Clinically proven barrier to pathogen ingress, yet well ventilated, breathable and comfortable.









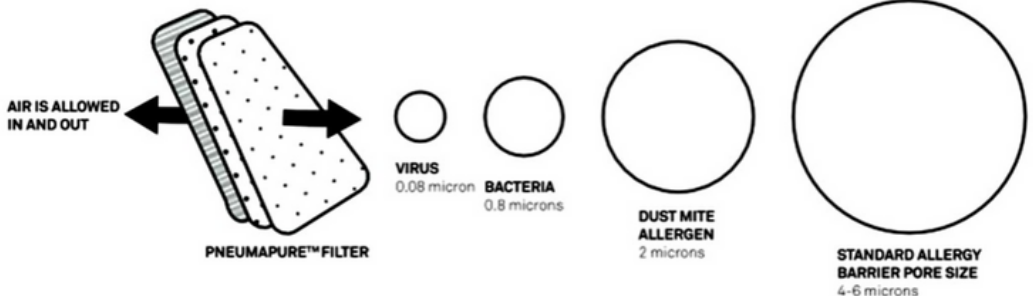


Patented bedding range, clinically proven to block viruses, allergens, microbes and mold.



Hermetically sealed cushioning device keeps pathogens out, because there is no stitching.



PosiPerle Cushions



Code	Description	Photos not to scale
PP8018	Mini Universal Cushion 35 × 22cm	
PP8017	Medium Universal Cushion 30 × 40cm	
PP8046	Large Universal Cushion 45 × 30cm	
PP8016	XL Universal Cushion 60 × 40cm	
PP8050	Small Delta Cushion 42 × 25cm	
PP8013	XL Delta Cushion 60 × 30cm	
PP8033	XS Cylindrical Cushion 45 × 15cm	
PP8015	XL Cylindrical Cushion 70 × 17cm	
PP8045	XS Semi Fowler Cushion 50 × 35 × 20cm	

PosiPerle Cushions

Code	Description	Photos not to scale
PP8014	XL Semi Fowler Cushion 70 × 56 × 14cm	
PP8049	Large Arc Cushion 30 × 78cm	
PP8048	XL Arc Cushion 35 × 90cm	
PP8047	XS Abduction Wedge 12 × 23cm	
PP8034	XL Abduction Wedge 30 × 20 × 15cm	
PP8035	1/2 Circle Cushion 43 × 13cm	
PP8051	Large Leg Wedge Cushion	
PP8012	Half Moon Cushion XL 180 × 30cm	
PP8011	Decubitus Cushion XL 180 × 30cm	

PosiPillow Cushions

Fibre Filled Cushions

Code	Description	Photos not to scale
PF100	PosiPillow Cushion Small 40 × 60cm	
PF101	PosiPillow Cushion Large 50 × 60cm	









Carewave Cushions

Microbeaded Positioning Cushions

Carewave Cushions

Code	Description	Photos not to scale
NEG0609	Heel Cushion XL 90 × 15cm	
NEG0880	Ring Cushion XS 106 × 45cm	
NEG0612	Ring Cushion XL 130 × 20cm	
NEG0885	Heel Pad XS One Size	
NEG0616	Heel Pad XL One Size	
NEG0732	Hemi Arm Cushion XL 48 × 35cm	
NEG0890	Bone Cushion XS 28 × 19 × 6cm	
NEG0919	Bone Cushion XL 34 × 24 × 10cm	
NEG1657	XXL Delta Cushion 50 × 70cm	
NEG1103	Ear Cushion 30 × 30cm	

Carewave Cushions








Code	Description	Photos not to scale
NEG2015	Support Wedge 20 × 45 × 15 × 7.5cm	
NEG1642	1/2 Circle & Cylinder 45 × 45 × 14cm	
NEG1075	Stablilegs XL - positioning kit composed of a base pad & 4 cushions	
NEG1168V	Palm Protector - Velcro 14.5 × 4.5cm	
F2299	Leg Through cushion - Foam	
NEG09646	Universal Cushion 40 × 10cm	
NEG0942	Segment Cushion 46 × 22cm	
NEG0941	Arc Cushion XL 190 × 34cm	

Flo-Form Fluidised Positioning Cushions

Multi Patient Use

Fluidised Technology

Flo-Form Cushions

Code	Description	Photos not to scale
FF7010	Neonatal Head Positioner 18 × 25cm	
FF9015	Neonatal Small Positioner 23 × 38cm	
FF1220	Neonatal Medium Positioner 30 × 50cm	
FF1824	Neonatal Large Positioner 46 × 60cm	
FF2028	Neonatal XL Positioner 50 × 71cm	
FU400	Utility Positioner 46 × 30cm	
FFS100	Adult Positioner 51 × 64cm	

Cotton & Bamboo Covers

Cotton and bamboo covers are available for the full PosiPerle range.



The bamboo cover shown is made of a more natural wicking, breathable material.



The covers are made with an easy access zip opening. Allowing the cushion to be taken out and the cover to be machine washed.



Cotton Cover

Head Support

Head Support

NEG0612 Ring Cushion XL

These cushions have been designed to relieve the patients head and neck at either a 30° or 130° angle



NEG0880 Ring Cushion XS



PP8051 Leg Wedge XL - Can also be used for head support



PP8015 Cylindrical XL



Head Support

PP8016 Universal Cushion XL



PP8014 Semi Fowler XL



PP8012 Half Moon Cushion XL



Head Support



PP8035 1/2 Circle Cushion



NEG0890 Bone Cushion XS
NEG0919 Bone Cushion XL



NEG1103 Ear Cushion

PP8048 Arc Cushion XL



PP8049 Arc Cushion XS



Chest & Side Lying Support

Chest & Side Lying



PP8012 Half Moon XL



PP8015 Cylindrical Cushion XL

PP8013 Delta Cushion XL



Chest & Side Lying

PP8016 Universal Cushion XL



PP8011 Decubitus Cushion XL




Arm & Hand Support

Arm & Hand Support



PP8034 Abduction Wedge

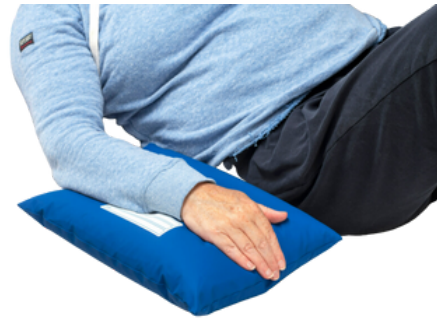

PP8017 Universal Cushion L



PP8016 Universal Cushion XL



PP8018 Mini Universal



NEG1168V Palm Protector

Leg Support

Leg Support



PP8014 Semi Fowler Cushion **XL**



PP8013 Delta Cushion **XL**

PP8015 Cylindrical Cushion **XL**



Leg & Knee Support

Leg & Knee Support



PP8051 Leg Wedge Cushion L



PP8016 Universal Cushion XL 23

Leg & Knee Support



NEG1657 Delta Cushion **XXL**



PP8045 Semi Fowler **XS**



NEG0947 Arc Cushion

Knee Support

Knee Support



PP8012 Half Moon Cushion XL

PP8015 Cylindrical XL

PP8015 Cylindrical XL



PP8016 Universal Cushion XL

Knee Support

PP8011 Decubitus Cushion XL



Knees and Abduction

Knees & Abduction



NEG0888 Abduction Wedge **XS**

PP8034 Abduction Wedge **XL**



PP8013 Delta


NEG1642 1/2 Circle
& Cylinder



PP8016 Universal Cushion &
PP8034 Abduction Wedge



Here is a combination
of the universal placed
under the legs and the
abduction between the
thighs

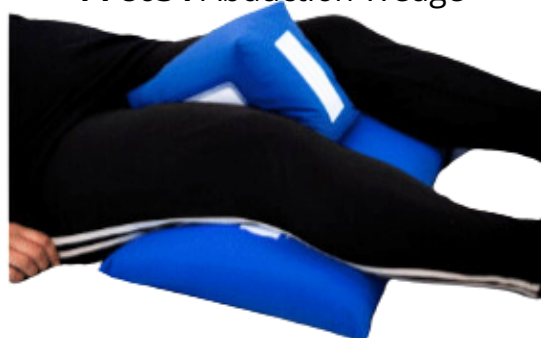

Knees & Abduction

PP8033 Cylindrical Cushion **XS**



NEG2015 Support Wedge



NEG0890 Bone Cushion **XS**



NEG0919 Bone Cushion **XL**

Knees & Abduction



PP8051 Large Leg Wedge

PP8035 1/2 Circle Cushion



PP8013 Delta XL

PP8035 1/2 Circle Cushion



PP8013 Delta XL

PP8018 Mini Universal

PP8015 Cylindrical



Foot & Heel

Foot & Heel

PP8015 Cylindrical



PP8033 Cylindrical



NEG0609 Heel Cushion



NEG1325 Leg
Wedge

PP8033 Cylindrical **XS**

PP8050 Delta **XS**

Foot & Heels

PP8015 Cylinder **XL** under knees

PP8033 Cylinder **XS** under heels



PP8016 Universal **XL**

NEG0616 Heel Pad **XL** in Blue

NEG0885 Heel Pad **XS** in Green



Foot & Heels

The Equinus Foot

Depending on how the patient compensates for the inability to bend the ankle properly as a consequence of Equinus foot, this may cause other conditions such as pressure sores forming on the ball of the foot or arch



NEG0609 Heel Cushion **XL**



Combination of **NEG0609** Heel Cushion and **PP8016** Universal **XL**

Foot & Heels



This shows the Delta **PP8013** under the knees, using the **NEG1642** as abduction cushions with the **PP8013** at the foot of the bed. The second example shows the **PP8050** – Delta **XS**.



To help prevent conditions such as Equinus foot, cushion such as our **PP8013** Delta **XL** or **PP8050 XS** can be placed at the bottom of the bed supporting the foot and heel.

The microbeads inside the cushion allows for the foot movement and reduces the risk of cramp

Stabilegs and Foam Through Leg Cushion

Stabilegs

NEG1075

The Stabilegs consists of a base cushion, 3 cylindrical cushions and one long cushion for the heels to sit on as you can see in the example.



The cushions are held in place by Velcro and the option of using the strap to support the legs



Foam Through Leg Cushion



F22996 Foam through leg cushion, great for leg alignment



Kit Bags

Kit Bags



Adult Positioner Kit



Paediatric Positioner Kit



Lateral Positioner Kit



Contracture Positioner Kit



Lower Limb Positioner Kit



Positioning Examples

Positioning Example Overviews



The Half Moon XL (**PP8012**) together with the Cylindrical cushion XL (**PP8015**)



The Half Moon cushion XL (**PP8012**) together with the Delta XL (**PP8013**) for knees flexion



The Half Moon XL (**PP8012**) together with the Semi Fowler XL (**PP8014**) for leg positioning and pressure relief on heel areas



2 x Half Moon XL (**PP8012**) and a Cylindrical XL (**PP8015**)

Positioning Example Overviews



1 Cylindrical XS (**PP8033**)
behind the head, 1 Half Moon
XL (**PP8012**) wrapped under
the legs and 1 Cylindrical XL
(**PP8015**) under the feet



Universal XL (**PP8016**) for
under the head, Delta XL
(**PP8013**) for back support
Universal S (**PP8017**) for
under the arm
Cylindrical XL (**PP8015**) for
between the legs



Universal XL (**PP8016**) for
under the head, Half Moon
XL (**PP8012**) for back
support and between the
legs and a Universal S
(**PP8017**) for the arm



Delta XL (**PP8013**) for side
lying support, Cylindrical XL
(**PP8015**) between the legs
and Universal XL (**PP8016**)
under the head

Positioning Example Overviews



Decubitus Cushion XL (**PP8011**)
supporting the back and
between the knees and a
Universal XL (**PP8016**) under
the head



Delta Cushion XL (**PP8013**)
support from the front of the
patient and a Universal XL
(**PP8016**) between the knees



Half Moon XL (**PP8012**)
supporting the head and
between the legs and a
Universal XL (**PP8016**) between
the ankles

Breathable mechanical **PneumaPure** filter allows clean air flow into the cushions, blocking out pathogens and allergens

Tested pathogens include:

MRSA

Aspergillus niger

C.difficile

Norovirus

E.coli


Influenza Type A

Pseudomonas aeruginosa

Adenovirus 5

Candida albicans

Coronavirus 229E (HcoV229E)

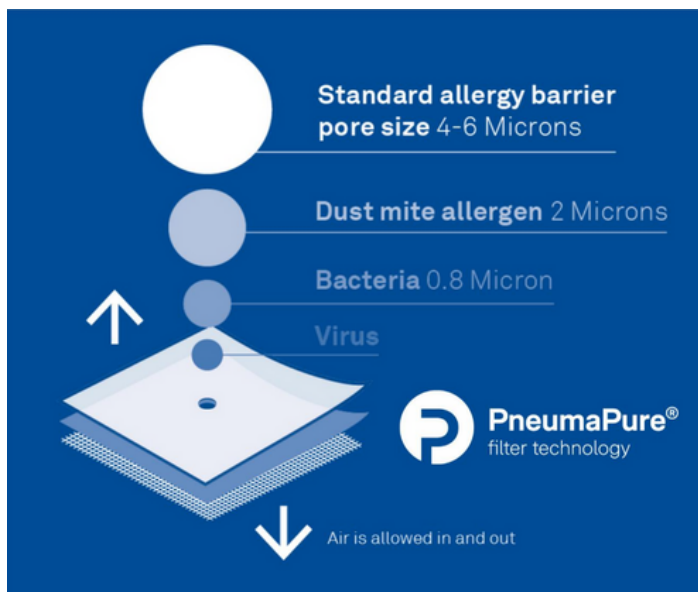

Features & Advantages



Durable, soft touch and high performance textile that blocks out contaminants. Vapour permeable and waterproof.

Heat sealed seams provide complete barrier - no stitching holes.

Breathable mechanical filter that allows clean air flow into the pillow but blocks out pathogens and allergens.

Certified as Class I Medical devices in the EU
Registered Class I Medical Device with the FDA
Approved and registered with the SFDA

Easy to clean, safe to use

For normal cleaning use mild detergent and water solution, avoid abrasive scrubbing of the filter and fabric;

Do not bleach, do not machine wash, do not dry clean, do not tumble dry, do not iron;

Follow instructions for the best result.



Flo-Form Fluidised Gel Positioners

Fluidised Positioners



FS100 Adult Positioner

- Contours and supports the body to offload the Scapula and Sacrum
- Maintains the Therapeutic Positioning
- Promotes neutral body alignment
- Facilities tube/line management
- Supports the neck and shoulders to offload the occiput
- Supports the Cervical Spine
- Promotes neutral Spine alignment
- Facilitates tube/line management
- Protects the ears, nose and face against shear and friction

With its fluidised technology, Flo-Form can be moulded to any shape and will hold its form until it is remoulded. With its unique characteristic, Flo-Form is an excellent medium for patient positioning and pressure redistribution.

Fluidised positioning devices (FPD's) are comprised of lubricated microspheres, giving them the ability to 3 - dimensionally contour to the patients specific shape

Neonatal Positioners

- Creates a soft outer boundary to support a flexed, midline position
- Helps maintain neutral body alignment and positive airway
- Encourages normal musculoskeletal and neuromotor development
- Recommended for single patient use



FF1824 Neonatal Large Positioner



- Redistributes pressure to protect the skin
- Helps reduce lateral skull flattening
- Promotes neutral head/trunk alignment
- Facilitates tube/line management
- Recommended for single patient use

FF7010 Neonatal Head Positioner




Weighted Tail Cushion Covers

Weighted Tail Cushion

Our 'weighted tail' cushion covers are cleverly designed to completely eliminate the seat cushion to slide on a riser recliner chair whilst the chair is being risen, enabling the cushion to safely stay in place for the user

- Additional cushion stays safely in in the chair when rising
- Designed with 'weighted tail' tabs which are welded into the seam of the cover
- Stretch vapour- permeable fabric
- Can be used with multiple pressure care cushions
- Heat welded seams and zipped for easy access

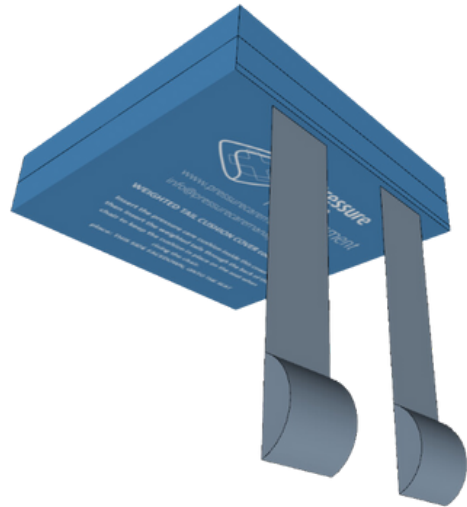



SPECIFICATIONS

Fitted under patient mattress

Automated Cycle Times

Compatible with profiling beds and most mattress types

Fits on beds 80 - 120cm wide

Max user weight of 320kg

Gently tilts the user throughout left-neutral-right

Auto deflate CPR function

20-25 degree cell inflation to redistribute body weight and offload pressure points

Intelligent Bluetooth connection to App to remotely adjust functions

Advanced buttons on pump faceplate including a pause button

3 year warranty



Easy fit under mattress



We offer bespoke quotes if required.
For all quotes and enquiries please email
info@pressurecaremanagement.com

Alternatively, you can visit our website for our full range of positioning
cushion and products.

Please note, we do provide VAT invoices with all orders, including orders
from the website.

We also offer a VAT exempt option via the website when purchasing
products.

UK and international shipping available

Pressure Care Management
Unit 2
Edison Gate
West Portway Ind Est
Andover
Hampshire
SP10 3SE

T: 01264 366330
E: info@pressurecaremanagement.com
W: pressurecaremanagement.com